DE

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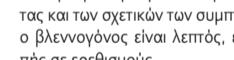
Composition (selon INCI) :

Aqua, Glycerin, Propylene Glycol, Hydroxyethylcellulose, Citric Acid, Levulinic Acid, P-Anisic Acid, Sodium Hydroxide

Made in Switzerland

Fabricant / Distribution CH :

Tenant AG, 4452 Itingen

Distribution D :

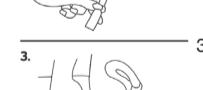
Tenant Deutschland GmbH,

53332 Bornheim-Walberberg

Distribution AT :

GYNCOM GmbH, 1210 Wien.

www.gynofit.com

UK

How Gynofit Moisturising Gel works

Gynofit Moisturising gel is an aqueous preparation that forms a thin Moisturising film on the vaginal mucous membrane.

Uses and purpose

For temporary relief of vaginal dryness and related symptoms, especially if the mucous membrane is thin, sensitive, and irritation-prone.

The Moisturising gel helps relieve vaginal dryness resulting from hormonal changes during pregnancy and following delivery, cancer therapy, menopause, etc. Moreover, it makes you feel better by alleviating symptoms like burning and/or itching. Gynofit Moisturising gel also reduces discomfort during sexual intercourse and hinders genital irritation.

Side effects

Generally you should not experience any side effects, if you apply Gynofit Moisturising gel as directed. However, the vagina's mucous membrane may become irritated or allergic reactions may occur if you are predisposed to such conditions, particularly if the mucous lining was damaged by improper care/hygiene, or from use of local contraceptives.

Contraindications

Do not use Gynofit Moisturising gel if you have ever had an allergic reaction to any of its ingredients. If you experience such reactions, stop use immediately and consult your healthcare provider.

Caution

You can use Gynofit Moisturising gel daily, as needed.

This gel is not a contraceptive, since it contains no spermicides, but is safe for use with condoms.

Contains no preservatives, parabens or perfume.

With LAG for a long-lasting moisture.

Keep out of reach of children and store at 15-25 °C.

See packaging for expiry date.

Composition (according to INCI):

Water, glycerin, propylene glycol, hydroxyethyl cellulose, citric acid, levulinic acid, p-anisic acid, sodium hydroxide

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Tentan AG, 4452 Itingen